



## Instrument maintenance

# SPEZIAL ÖLSPRAY

Spray for maintenance of instruments

- Based on medical-grade white oil
- For hand-piece maintenance and lubrication prior to steam sterilization
- Optimal lubrication of moving instrument parts



### Product description

**SPEZIAL ÖLSPRAY** is an aerosol spray for the maintenance of medical instruments, especially for articular instruments. For the maintenance of hinges and valves of rigid endoscopes. The special spray valve provides a fine, even protective layer. A subsequent sterilisation after the application on the instruments is not affected by this protective layer. **SPEZIAL ÖLSPRAY** preserves the value of your instruments because of an optimal lubrication and an effective corrosion inhibition. Due to the use of paraffinum liquidum, **SPEZIAL ÖLSPRAY** is compatible with most materials and toxicologically harmless according to drug and food law.

### Range of application

For maintenance of all kinds of stainless steel medical devices and surgical instruments. For maintenance of handpieces and for lubrication prior to steam sterilization.

### Application

Spray the halfopened instruments with **SPEZIAL ÖLSPRAY** from a distance of about 25 cm. If indicated sterilise the instruments afterwards.

### Attention:

Do not apply too much oil, in order not to interfere with sterilisation. Only a thin film should remain.

### Hinges, turbines or handpieces and angle pieces:

Use **Spezial Ölspray** two times a day or spray it through the devices for app. 1 sec before sterilization or autoclavation respectively. To apply the product hold the spray bottle vertically. Shake before use.

### Application notes

Always maintain this distance and spray on a thin oil film only. Shake well before use. If **SPEZIAL ÖLSPRAY** is used on latex or rubber these materials may swell. In these cases use **Silikonspray**.

### Composition

Paraffinum liquidum  
Propellants: propane/butane (free from CFC's)

### Precautionary and hazard statements

Extremely flammable aerosol. Pressurised container: May burst if heated. Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. Do not spray on an open flame or other ignition source. Do not pierce or burn, even after use. Protect from sunlight. Do not expose to temperatures exceeding 50 °C/122 °F. Dispose of contents/container to approved disposal company or local collection.

For professional use only.

Product	Single unit	Content	Delivery unit	REF
SPEZIAL ÖLSPRAY	aerosol can	500ml	12	00-108-005

